


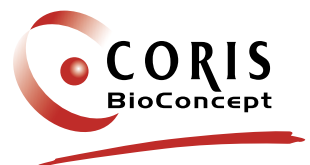

Proguanil-Strip


Rapid diagnostic test for *in vitro* detection of Proguanil in urine

- Easy to read
- Rapid (10')
- Specific
- Sensitive
- No equipment required
- No dilution



www.corisbio.com

Proguanil-Strip

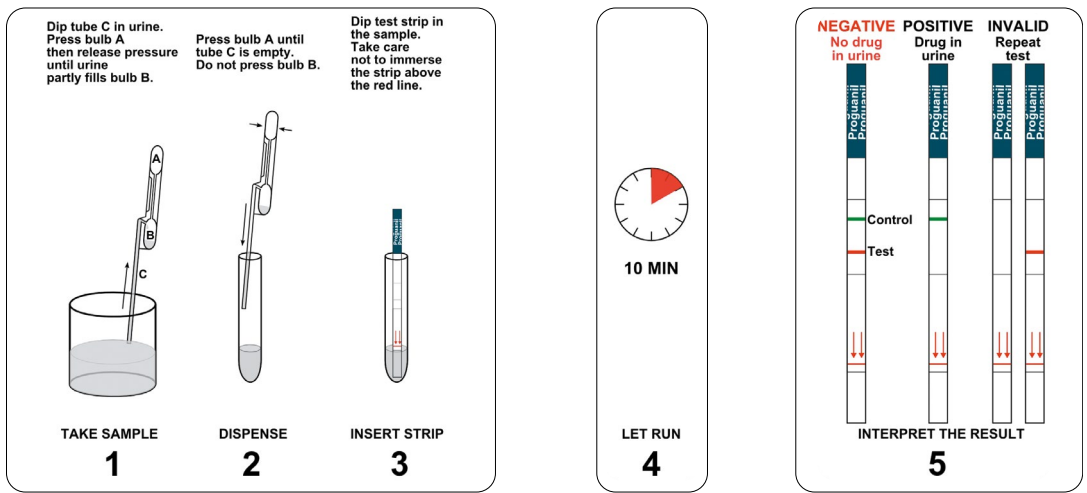

Performance of the kit

228 urines containing Proguanil have been collected and tested on the spot. All PG-strip show the presence of Proguanil in urine.

The presence of Proguanil in 170 of these urine samples was confirmed in France by HPLC (High Performance Liquid Chromatography).

The detection limit of Proguanil ranges from 20 to 80ng/mL.


For more technical information

client.care@corisbio.com

Ordering Information

PRODUCT NAME	DESCRIPTION	STORAGE	CODE
Proguanil-Strip	20 tests/kit	4°C to 30°C	C-10T1

— Distributed by —

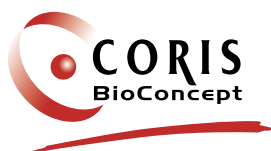

CREALYS Science Park
Rue Guillaume Fouquet 11
5032 Gembloux - BELGIUM
T +32 (0)81 719 910 - F +32 (0)81 719 919
sales@corisbio.com - www.corisbio.com

MADE IN BELGIUM

D-CUS-10-T1-S/04
March 2023