


K-1516



GastroVir K-Set


**Rapid diagnostic test for *in vitro* detection of
Rotavirus & Adenovirus (serotypes 40/41) in stool
specimens**

- Easy to use
- Easy to read
- Specific
- Sensitive
- Reliable
- Rapid (10')

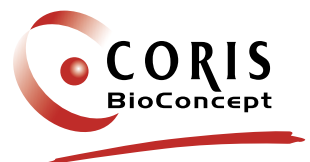

www.corisbio.com

GastroVir K-SeT

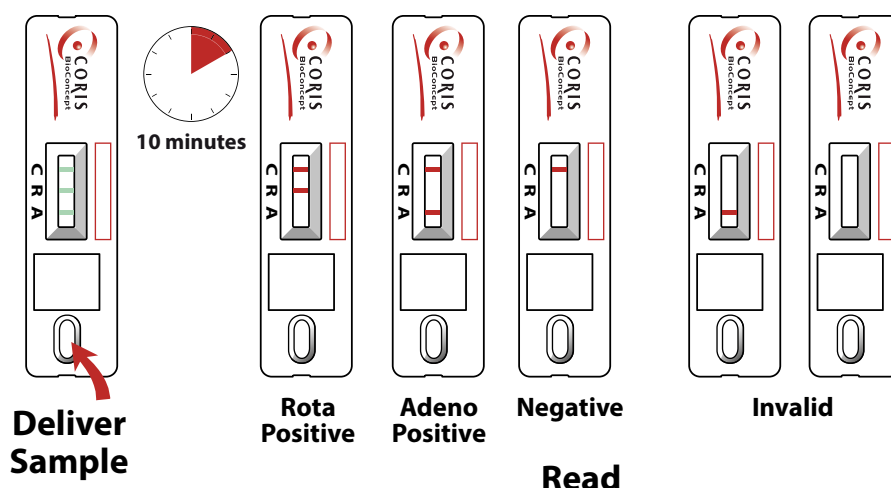

Performance of the kit


| | Rotavirus detection Compared to PAGE method (n=400) (95% confidence interval) | Adenovirus 40/41 detection Compared to PCR method (n=400) (95% confidence interval) |
|--------------------|--|--|
| Sensitivity | 97.4% (92.1 to 99.3%) | 96.7% (80.1 to 99.8%) |
| Specificity | 97.9% (95.2 to 99.1%) | 97.3% (94.9 to 98.6%) |
| PPV | 95.0% (88.9 to 97.9%) | 74.4% (57.6 to 86.4%) |
| NPV | 98.9% (96.7 to 99.7%) | 99.7% (98.2 to 100%) |
| Agreement | 97.8% (391/400) | 97.3% (389/400) |

For more technical information

client.care@corisbio.com

Ordering Information

| PRODUCT NAME | DESCRIPTION | STORAGE | CODE |
|--|---|----------------|-------------|
| GastroVir K-SeT with collection set | 20 tests, 20 faecal sampling systems | 4°C to 30°C | K-1516 |

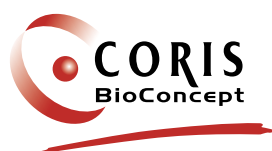

CREALYS Science Park
Rue Guillaume Fouquet 11
5032 Gembloux - BELGIUM
T +32 (0)81 719 910 - **F** +32 (0)81 719 919
sales@corisbio.com - www.corisbio.com

— Distributed by —



MADE IN BELGIUM

D-CUS-10-16-K/08
March 2023